

# Nests



Created for the sensitive, hard to manage infant

- Decrease decels and desats
- Reduces stress and negative reactions and set-backs
- Gel beads push back for spatial & brain development
- Bags support baby's back, front or side
- Provides for proper positioning
- 3 sizes: Preemie, Newborn and Ultra-Preemie to meet the needs of all age and size babies

Order #	Product Description	Cs Qty
<b>Nest Sets Infant Positioning Device - Bead-Bags Only</b>		
09B	Preemie Nest Bead-Bags - Set of 3 Bags 2 Large Bags 12.5" x 4.75" x 1.75" 1 Small Bag 7.5" x 4.75" x 1.75" For infants under 36 weeks gestation	10 Sets
09BB	Newborn Nest Bead-Bags - Set of 3 Bags 2 Large Bags 13.5" x 5.25" x 1.75" 1 Small Bag 9.5" x 5.25" x 1.75" For infants over 36 weeks gestation	10 Sets

Order #	Product Description	Cs Qty
<b>Nest Pads Only</b>		
09A	Preemie Fleece Nest Pads 30" x 27"	12
09AK	Preemie Knit Nest Pads 30" x 27"	12
09AAD	Preemie/Newborn Disposable Pads 30" x 27"	20/Pkg
09AA	Newborn Fleece Nest Pads 36" x 29"	12
09AAK	Newborn Knit Nest Pads 36" x 29"	12

## Nest Set-Up

Nest Pads are designed with ribbed knit sleeves sewn onto backside of pad. Bags are filled with rebound able, super soft gel beads; 2 long bags and 1 short bag. Each sleeve has Velcro at each end to keep the bags from falling out. Turn pad over and place bags in rectangle as shown.



Super soft gel beads



## How to Use and Turn Infant and Nest in Seconds

Place baby supine in the center, then grab pad and bead-bag close to you. Roll bag and pad toward you to turn baby from supine to side position. Move bead-bags to interface with baby and surround baby with womb-like feeling of soft, cushy, rebound. No decels, no desats... Just calm. Turning and Position in seconds!!

